

# PL1 Storage Shelf Instructions

**PL1/SHELF**

The PL1 Storage Shelf, Part No. PL1/SHELF, is an easy-to-fit mounting accessory that safely and securely holds a PL1 portable loop amplifier when not in use.

The shelf is wall-mounted and normally in plain sight of hearing aid users who can request the PL1 amplifier be brought to a counter/tabletop, when required.

Shelf features include a pre-cut hole for feeding through the power supply unit (PSU) lead to the PL1's charger socket, allowing the PL1 amplifier to be charged whilst stored on the shelf.



## PL1/SHELF Features

PSU lead shown fed through pre-cut hole in top of shelf

Use built-in pocket to store PL1 instruction booklet

Use pre-cut hole to feed through PSU lead and secure using cable tie (supplied). Ensure the PSU lead will connect to the PL1 charger socket before securing.

Recess for label

PSU lead shown fed through pre-cut hole in bottom of shelf

Use three fixing holes to wall-mount the shelf. Ensure PSU lead is not trapped when mounting shelf.



PL1/SHELF Specification	
Material:	Black, High Impact Polystyrene (HIPS)
Weight:	390 g
Dimensions:	263 mm (H) x 260 mm (W) x 140 mm (D)
Fixing Holes:	3 off holes, each 5 mm max. diameter. Use screws compatible with fixing to a wall.